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Thank you for purchasing one of our mattresses and/or accessories. We wish you pleasant sleep and a proper rest.

PASSIVE ANTI-DECUBITUS MATTRESSES

Mattresses (including their variants) in the Hilding Anders Tropico, CUREM and SPIRIT ranges are constructed to promote pressure ulcer prevention, to relax parts of the body and for relaxation. The mattresses can be used at home and for hospital and institutional care. They are designed exclusively for indoor use.

- **Swisslab Active, Swisslab Natur, Swisslab Big Boy, Super Fox Visco, Super Fox Blue, Super Fox Cloud,**
- **Klára Tropico, Latex Supreme, Alpine Blue Air, Kolos Bio Ecology, Amundsen, Austin Air Latex, Austin Air Visco, Austin Air Geltouch, Arabela, Arabela Hard, Kontakt Bambus, Kolos, Romantika Kašmír, Heureka Plus, Dáša Tropico, Šárka Tropico, Nika, Enigma, Favorit, Maxi, Petra Tropico, Riviera Plus, Tropico Guard Air Hybrid, Tropico Guard Antibacterial, Tropico Guard Medical, Tropico Guard Ortho, Fest XL, Partner Bamboo XL, Sol, Airspring Bluegel, Targa Flexi, Atlas**
- **Curem C2000, Curem C3000 HY, Curem C3500, Curem C4500, Curem C7000 XD, Curem Dream Forte, Curem H6 Robust, Curem H6 Velvet**
- **Spirit Superior Latex, Spirit Superior Visco, Spirit Superior Triumph, Spirit Superior Cloud, Spirit Superior Nucleus, Spirit Superior Comodore,**
- **Topper Visco, Topper Flexi; Topper Guard Medical Aero, Topper Spirit Superior Root, children's mattresses: Dana, Vojta a Vojta Travel**

Passive anti-decubitus mattresses of the Tropico, CUREM and SPIRIT series are intended for the treatment of pressure ulcers, including their prevention (depending on the type) for long-term use. The mattresses are designed as Class I medical devices.

MATTRESS USE AND MAINTENANCE GUIDE

Thank you for purchasing one of our mattresses.

Mattress dimensions: as per the offer or agreement with the retailer. The size tolerances are indicated below, in accordance with valid standards.

The manufacturer retains the right to colour deviations in the materials used that do not affect the performance of the product, in particular differences in the colour of the foams, cover materials, and zips compared to the sample, as well as the designs on the covers.

Differences in firmness, determined by the individual foam manufacturers, are also permissible.

Recommended mattress care:

Ventilation – keep in mind that mattresses absorb body moisture. It is important that the mattress is thoroughly ventilated and, if possible, kept uncovered. This will ensure a longer life-span for the materials.

Vacuum cleaning – once a month is recommended.

Cleaning – as needed using dry foam or waxing (wet foam with reduced moisture followed by drying). You can also lightly and evenly apply bicarbonate of soda, leave it to act for several hours, and then thoroughly vacuum.

Mattresses are equipped as standard with a knitted or woven breathable textile cover (its washability, composition and construction differ depending on the specific mattress type); they may also be equipped with a coated polyurethane (PU) cover; this coating is impermeable to liquids yet remains permeable to vapour, enabling both more effective home care and making this variant suitable for professional use in healthcare, nursing etc. The cover is non-slip, suitable for allergy sufferers and asthmatics, can be wiped down and is washable at high temperatures (up to 95°C). It has a double-sided zip (“U” shaped for easy removal and application), protected by a flap to keep it clean and prevent the ingress of dirt to the mattress core.

Read the cover label that provides care symbols. Do not use a fabric softener. Never iron covers, and do not use caustic, corrosive, or abrasive substances or objects to remove stains and dirt! Always proceed in accordance with the care symbols sewn into the hem of the cover (near the zip). Covers cannot be tumble dried. Only use your mattress with the original cover, if possible, otherwise we cannot guarantee optimum functionality. Clean or wash the mattress cover immediately after contamination or if it gets dirty. You can also use a spare cover, which can be purchased separately. Make sure that no foreign object gets between the mattress, respectively the mattress core, and the cover, or between the mattress the patient’s skin!

We recommend using a mattress protector to extend the product’s lifespan. Turn over the mattress regularly twice a year. Let compressed mattresses rest without loading after removal from packaging for at least 72 hours. These compressed mattresses can be stored in a compressed state for a maximum of 30 days. Uncompressed mattresses can be used immediately after unpacking. Immediately after replacing a mattress there may be sudden back pain caused by the spine aligning. Our mattresses may give off a characteristic aroma after being unpacked, however this is completely safe and will disappear after a few days. The materials used are safe and hygienic.

Note for mattresses and toppers that include memory foam (also known, for example, as visco, viscoelastic, lazy, or thermoactive foam): The unique properties of memory foam (e.g., Curemfoam™) mean that temperature changes during transport might cause slight deformation/shrinkage (in particular when delivered to heights). After unpacking your mattress, leave it to “breathe” at room temperature for around 24 hours – we also recommend giving it a slight shake. The mattress will return to its original size after it reaches room temperature.

Mattress placement (base):

To ensure your mattress provides ideal orthopaedic properties and optimal airiness, it is important to select a suitable base that corresponds to the mattress construction. This will also ensure the longest possible lifespan for your mattress. Make sure you select the correct mattress size, and before unpacking the mattress check that the size corresponds to the internal dimensions of the bed. Mattresses for hospital care are designed to be used with standard adjustable beds (the positioning options of the mattress may be reduced if the bed has fixed side panel). Either an adjustable hospital bed (medical device) or a regular bed may be used for home care if this meets the user's requirements. By selecting a quality base (slatted frame, bed) you will ensure proper ventilation and the longest possible mattress lifespan. Sprung mattresses must be used on a solid base or slatted base where the gaps between the slats do not exceed 4 cm. Sandwich and latex mattresses are suitable for use on a sprung slatted base, which supports the orthopaedic properties of the mattress, but they can also be placed on a slatted frame. A sprung slatted base must always have at least 24 sprung slats, and the gaps between them must not exceed 4 cm! Foam and latex mattresses (except ones reinforced with coconut/latex board) are suitable for adjustable frames, whether mechanical or motorised. Do not place mattresses on wire or metal frames unless stated otherwise on the packaging leaflet or mattress label. Place mattresses with a memory side intended for professional use in healthcare so that this side is always uppermost (the preferred side of the mattress for use). With double-sided mattresses (where the sides have differing firmness and/or construction), both sides may be used depending on the need for firmness/the resulting pressure-relief effect. If a mattress does not have a firmer/ softer side, respectively it is not equipped with a single-sided memory foam layer, always place the mattress (again for professional use, as indicated above) with the profiled (trimmed) side up!

Transport, storage, basic conditions of use:

Mattresses are packaged in a tightly sealed low-density polyethylene (LDPE) wrapper, and possibly in a cardboard box or on a pallet depending on the customer requirements, unless this is contrary to the regulations for the storage and transport of this type of product. Be as careful as possible when removing the packaging to ensure you do not cut too deeply and damage the mattress! Never fold the mattresses; mattresses with pocket springs must be transported and stored horizontally. Store in a dry place at a temperature of over 5°C and protect from freezing. Do not expose mattresses to direct sunlight, even if they are in their original packaging, and also avoid sudden differences in temperature and humidity. If you place the product in an overly dry environment you should let it acclimatise – again, avoid sudden changes in humidity. Store mattresses and other products of a similar nature away from sources of ignition, and do not dispose of them in fire! The materials used in our mattresses meet standard requirements for reduced flammability (for more information see the label for the specific mattresses, describing either the standard or stricter requirement for fire safety), however the materials used are generally considered flammable and so the indicated conditions for use must be complied with. Please use a powder or foam fire extinguisher to put out small, local areas of burning! Dispose of your mattress only at designated places – mattresses are disposed of through landfilling, while their individual components are recyclable. Keep the packaging out of the reach of children! Dispose of in accordance with the Waste Act. The mattress upholstery 22 layer – single- or multi-layer – is made of LDPE (low-density polyethylene) and is recyclable. Mattresses with RE foam must only be stored in a horizontal position.

GENERAL WARRANTY CONDITIONS

For hygiene reasons, a purchased mattress cannot be exchanged for a different type or re-turned once used. In the event of a defect in a purchased product during the warranty term, the customer is entitled to make a warranty claim. A defect means a change in a product (its properties) caused by a manufacturing defect or a defect in the materials used to manufacture the product. The following, among other things, cannot be considered a defect: a change in a product (its properties) that occurred during the warranty term as a result of its usual wear and tear, warping of the upholstery layer or a change in the size of the upholstery layer caused by the normal use of the mattress, or increased sweating on the mattress (in particular for mattresses with memory – lazy – foam, which inherently has excellent thermal accumulation and can thus cause an increased feeling of heat and thus sweating).

The following also cannot be considered a defect:

- Size within the permitted tolerance of +/- 10 mm; Mattress size tolerance when placed on a base/ bed (e.g. a bed with non-recessed mattress): width and length +/- 20 mm; height +/- 10 mm. The permitted tolerance in the firmness of the mattress core/foam is +/- 15% (compression resistance in kPa). A mattress with pressure-bonded (recycled) "RE" foam has, due to the manufacturing technique of the foam, a permitted firmness tolerance of +/- 20%.
- Sagging (respectively deformation and changes in properties), which are within the permitted tolerances for mattresses made from polyurethane (Flexifoam®, PUR, cold-pressed, visco, hybrid etc.) foams, especially sagging that does not exceed 15% of the total height of the mattress.
- Colour changes in the foam and upholstery layer fabrics compared to the fabric pattern and fabric upholstery layer designs or patterns of the stitching of the upholstery layer fabrics that do not affect the performance of the mattress.
- The manufacturer reserves the right to make any changes in the technology of the product, its construction, colouring, design, etc. without prior notice, while the appearance of the product may differ from that in promotional and illustrative photographs of the product. A mattress with dimensions over 90x200 cm may be delivered with glued structural joints. Due to the standard width of fabrics, mattresses of over 90x200 cm may be delivered with a different cover stitching pattern that does not affect the performance of the product. A block of high-quality hybrid foam is glued to all types of mattresses (including latex) when lengthened—meaning with a size of over 200 cm.
- For other covers than PU - impermeable (in particular those with a high polyester fibre content), we do not recommend using terry and cotton bed linen, as this can result in pilling. We always recommend the use of a mattress protector in such cases.

Warranty card, general warranty conditions:

A warranty for defects for products of Hilding Anders Česká republika a.s. is provided to the buyer in accordance with generally valid legislation of 24 months, potentially extended in the cases indicated below. The basic requisite for a proper warranty claim is demonstration of the purchase of the product from a retailer, ideally through the presentation of an appropriate document (receipt, invoice, etc.) and the presentation of the product that is the subject of the warranty claim with a description of the defect, and potentially the warranty card (especially for warranties of over 24 months). For this reason, please be sure to keep - for any eventual warranty claim - the warranty card, the document demonstrating the purchase of the product, and the product label, which is part of the packaging of every product. The buyer must carefully check and inspect the product at collection. If they discover a defect, they must notify the re-tailer of this fact without delay. The manufacturer bears no liability for defects caused through the transport of a product using the buyer's own means of transport, or for a later claim against a delivered quantity or damage to a consignment that was not indicated on the document on the receipt of the products.

The contact information needed for making a warranty claim is listed on the receipt. In the event of a warranty claim, please proceed in accordance with the Retailer's Warranty Conditions, which you will find on its website or directly in their premises.

Statutory warranty:

Provided that the product has been properly used and maintained in accordance with these instructions, the right to make a warranty claim may be exercised by the buyer within 24 months (unless indicated otherwise) from acceptance of the product when purchased from a retailer; and if the product was not collected from a retailer, acceptance of the product means the moment the buyer accepted the product from a forwarder. The term of the right from defective performance (hereinafter the "warranty term") will be extended by the time the product undergoes warranty repairs, meaning until such time as the buyer is obliged to collect the product after the completion of the warranty proceedings. The warranty term must not be confused with the product lifespan, meaning the time the product, considering its properties, purpose and differences in intensity of use, should last with proper use and treatment, including maintenance. The warranty term is a general administrative term that has no direct connection with the usual lifespan of a specific product. The right of the buyer to exercise a warranty claim expires with the expiration of the warranty term. The warranty does not apply to normal wear and tear to a mattress caused through its normal use. Further, liability for defects does not apply to any defects indicated to the buyer by the retailer at the time of purchase (conclusion of the contract) if the buyer did not refuse to accept the product. Further, the rights from a defective performance cannot be exercised for products sold at a lower price, where this lower price was arranged precisely due to defects the buyer was familiarised with before purchasing the product.

Extended warranty:

The manufacturer provides the buyer with an extended warranty for defects to the mattress core for selected mattresses under the conditions below, while the term of the warranty for defects is indicated on the label or sticker on the mattress packaging. An extended warranty term for defects to the mattress core is provided for selected mattresses, conditional on the placement of the mattress on a flexible base (sprung slatted base), with the exception of sprung mattresses. This benefit is free of charge – you only need to register your mattress to obtain this extended warranty without cost.

It is important that the buyer registers their mattress no later than 30 calendar days from the purchase date at <https://zaruka.matracetropico.cz>

To register you only need to copy the name, size and serial number (SN code) – you will find this on the sticker located on the shorter side of the mattress. You can obtain more details about the extended warranty from the retailer.

The extended warranty for defects only applies to the mattress core, while if the mattress core is replaced, the term of the warranty on the new core is the remainder of the extended warranty, however at least 24 months.

Extended regressive warranty for Curem and SPIRIT mattresses:

The manufacturer provides customers with an extended “Regressive” warranty for mattress core defects for selected Curem and SPIRIT mattresses of up to 10 years, as indicated on the label or sticker on the mattress packaging. This extended warranty always only applies to the mattress core. From 0 to 6 years, the customer has the right to full payment (meaning repair or replacement), while this is reduced by 20% per year after 6 years.

This extended warranty is also conditional on the registration of the buyer and the mattress at <https://zaruka.matracetropico.cz>.

By purchasing the product – mattress – the buyer confirms that they were properly familiarised with the warranty rules of the retailer.

Mattresses for which warranty claims are being exercised must be complete and whole, and must be in a hygienically acceptable condition (meaning clean, dry, hygienically safe, etc.), if possible in their original or in replacement packaging that is suitable for transporting the product (note – stretch film is not considered suitable packaging for transport!). The product must not be folded or rolled up, except for compressed mattresses. If the product that is the subject of the warranty claim is unhygienic (dirty, wet, smelly etc.), the warranty claim will be suspended until such time as the buyer rectifies this situation, while this fact will be entered into the claim report. If, however, the buyer does not present a hygienic mattress for the warranty proceedings within 1 month from the date of suspension, the warranty claim will be terminated for reason of being unjustified.

**Manufacturer: Hilding Anders Česká republika a.s., Branská 329, 514 01 Jilemnice.
Manufactured in the Czech Republic.**

Date of issue of the last revision: 2 September 2022, rev. 6

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